Saliva Collection Kit

EN

Intended use

The kit provides all components for collection and safe transfer of saliva to a laboratory for analysis. After spitting in the empty tube, a blue fluid (InActiv Blue™) must be added to make the sample non-infectious and to stabilize the RNA for at least 30 days. Extensive information about InActiv Blue™ can be found on www.inactivblue.com.

Product codes

IB_COL: 1x collection funnel and tube,
2 ml InActiv Blue™, Alcomed pads

Adverse events / incidents

Notice to FertiPro NV (ra@fertipro.com) and to competent authority of your country (for Belgium: inform FAMHP; www.famhp.be).

Warning

Do not drink the blue fluid in any way and wash your hands or other skin exposed area after contact with the fluid. When spills occur, clean up with water (do not use bleach).

If you are a responsible adult who helps with saliva collection of another individual, make sure that you use the required personal protection. Follow the procedure for droplet protection: use gloves, coat, surgical mask and eye protection (goggles or face shield).

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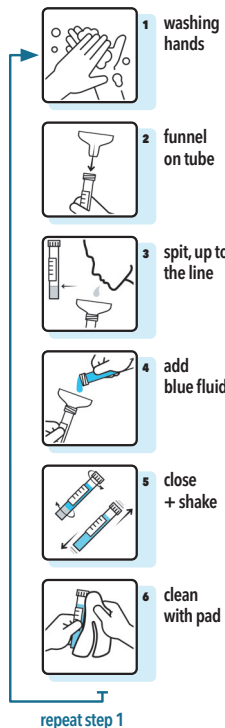
Please read the following instructions before collection

- Do not eat, drink, smoke, use mint or chewing gum for at least 30 min before saliva collection! If possible, rinse your mouth with water and wait for 10 minutes.
- Wash your hands thoroughly with water and soap. Never touch the inner side of the tube and funnel!
- Unscrew the cap of the empty tube and place the funnel on the tube, save the cap for later.
- Scrape throat and spit into the device until the amount of liquid saliva (not including bubbles) reaches the indicated line on the tube.
- Place the device with funnel and the collected saliva on a flat surface in a upright position.
- Open the tube with the yellow cap and pour the blue fluid via the funnel into the saliva tube.
- Remove the funnel and close the tube firmly.
- Shake the closed tube 10 seconds to mix the saliva with the blue liquid.
- Clean the outside of the tube and cap with the Alcomed pad.
- Ready for transport to the lab.
- Wash hands thoroughly with water and soap.

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